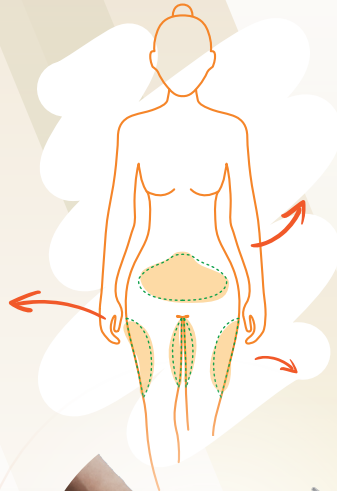
Lipomine.

User Manual
KULLANMA KILAVUZU

use the power in you

1

Preparation

- EN Disinfect the donor site and unwrap the sterile dressing, carefully place the dressing over the aspiration area.

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2

Infiltration

- EN Make an incision with the help of the plunger needle. Infuse the aspiration area with Tumescent / Klein Solution. Wait for 5-7 minutes for the solution to take effect.

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3

Lipoaspiration

- EN Attach the Lipoaspiration cannula to the 30cc Vaclok Syringe. Once the cannula is inserted through the donor site incision so that it is parallel to the adipose tissue layer, pull and lock the plunger with the help of the locking fin mechanism to create a sustained negative vacuum. Perform the aspiration through a series of back and forth motion. Pay attention to the location of the tip of the cannula as accidentally pulling it outside the body will result in vacuum loss.

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Decantation

- EN Position the fat filled 30cc VacLok syringe upside down onto the decantation apparatus for 10 minutes in order for the tumescent solution to settle at the bottom.

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Getting rid of the Tumescent Solution

- EN Discard the tumescent solution settlement at the bottom of syringe.

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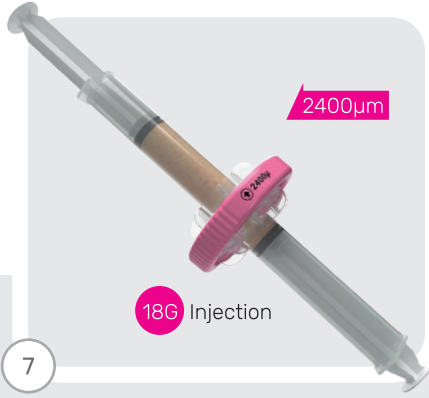
Transfer

- EN Connect the VacLok syringe to one side of the Luer Lock and attach the 20cc syringe to the other side then transfer the decanted fat.

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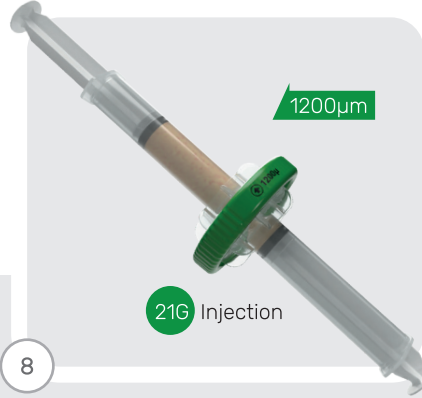
2400µm Microlyzer

EN Fix the 20cc adipose tissue filled syringe onto the base of the arrow displayed on the 2400µm Microlyzer and fix an empty 20cc syringe onto the other side. Make at least 5 passages, one passage being a full push of the condensed tissue through the blade from one side to another until nothing remains in the syringe on which the pressure is applied. Keep in mind that the last passage has to end on the side indicated by the arrow display on the Microlyzer, in other words opposite to the initial push. At the end of the process, the refined adipose tissue is transferred into a 1cc injection syringe, the end product is injectable through a 18G needle or cannula. Should further micronizing of the adipose tissue be required then move on to the next step.

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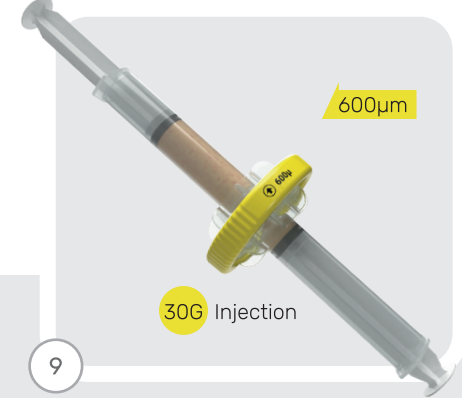
1200µm Microlyzer

EN Fix the adipose tissue filled syringe processed through the 2400µm blade onto the base of the arrow displayed on the 1200µm Microlyzer and fix an empty 20cc syringe onto the other side. Make at least 5 passages, one passage being a full push of the condensed tissue through the blade from one side to another until nothing remains in the syringe on which the pressure is applied. Keep in mind that the last passage has to end on the side indicated by the arrow display on the Microlyzer, in other words opposite to the initial push. At the end of the process, the refined adipose tissue is transferred into a 1cc injection syringe, the end product is injectable through a 21G needle or cannula. Should further micronizing of the adipose tissue be required then move on to the next step.

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600µm Microlyzer

EN Fix the adipose tissue filled syringe processed through the 1200µm blade onto the base of the arrow displayed on the 600µm Microlyzer and fix an empty 20cc syringe onto the other side. Make at least 5 passages, one passage being a full push of the condensed tissue through the blade from one side to another until nothing remains in the syringe on which the pressure is applied. Keep in mind that the last passage has to end on the side indicated by the arrow display on the Microlyzer, in other words opposite to the initial push. At the end of the process, the refined adipose tissue is transferred into a 1cc injection syringe, the end product is injectable through a 30G needle or cannula.

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